Duschklappsitz, wandmontiert „DKS 130“
Folding shower seat, wall-mounted

Art.Nr. / REF: 550 700 100

CE

Vorwort – Foreword

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Dear users,

This instructions helps you and the careers as a basis for sure, safe use of the Folding shower seat and a sensible use of the product possibilities.

The instructions describes the preparation, service, use, servicing and cleaning of the product.

This instructions is to be perused carefully and must be always accessible to all users of the product.

The assembly of the folding shower seat may occur only through the specialised dealers, under use of the subsoil to suitable assembly material.

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Standing insecurity / standing inability with use decrease / of missing functional ability of the lower extremities or other illnesses who are affecting the stability (e.g., strength loss with evil illnesses, chronic, advanced heart circulatory illnesses and lung illnesses) infers with a body position weakness, so that an independent use of the shower is still allowed.

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This product is suitable for reapplication. With the processing the standards of the manufacturer must be considered. This standards are made available on inquiry.

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The product contain no dangerous materials. After the product lifetime you supply the single parts according to the material call sign of your local disposal. The right and entire disposal can be taken over from the manufacturer.

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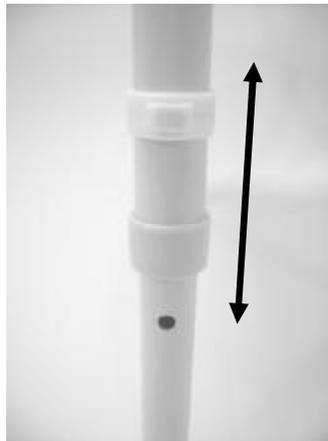
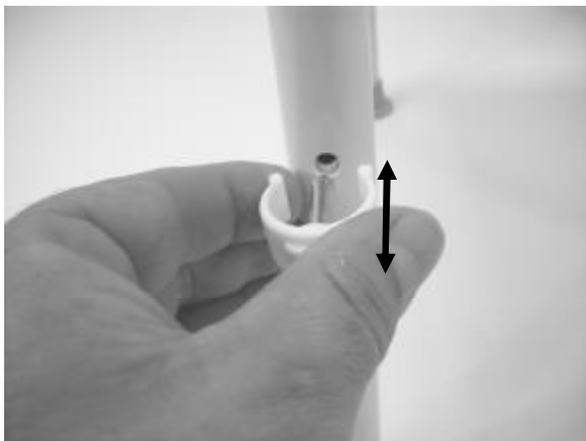
ATTENTION: *The mounting material is not added. The choice of the connection material is depending on the wall state. Needed are: 8 screws and 8 washers from stainless- steel as well as 8 to the masonry suitable raw plugs.*

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To adjust the footrests in the height first unlock the fixing clamp. Then the footrests can be shifted in 4 steps in the height for seat heights from 465, 490, 515 and 540 mm.

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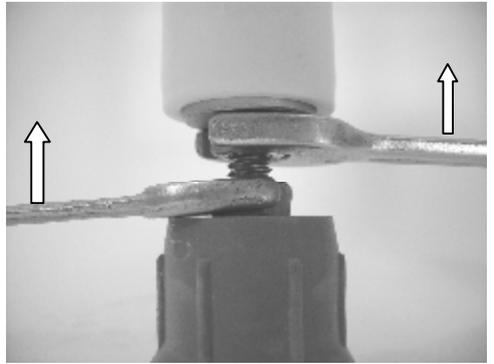
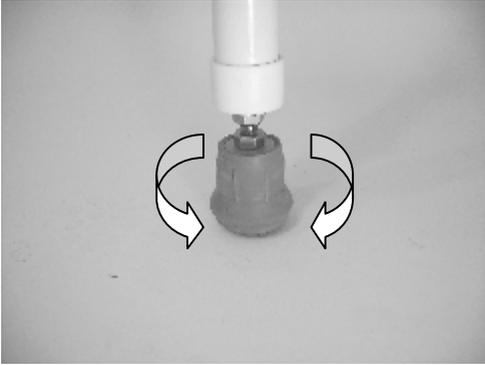
Important !

- *Both feet of the user must stand on the ground while sitting*
- *Take a normal chair on which the user comfortably can sit.*
- *Place the user on the chair.*
- *Measure the height of the seat.*
- *Choose from the adjustable masses of the Folding shower seat (465, 490, 515 or 540 mm) the measure to the ascertained measure is the next and transfer this measure from the ground of the shower tub on the wall.*
- *Cut a view window around the centre line from the drilling jig*
- *Put the drilling jig on the wall and transfer the 8 drillings. If possible, use the tile seams for drilling so as not to damage the tiles.*
- *Drill the holes and insert the raw plugs.*

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To compensate possible unevenness's in the shower 1 foot is equipped with a fine adjustment screw (part 6). Adjust the foot at the ground by turning in or out and lock with both nuts as illustrated.

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To unfold or fold the shower seat when not in use please always use the seatplate as handle (picture 1 + 2), never do it as shown in picture 3 – risk of finger pinch !

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Clean regularly by hand with a damp cloth and usual commercial cleaning agents!

Use spraying disinfectants p.e. Descosept AF or Bacillol AF.

Do not use any aggressive detergents!

Screws of DKS 130 must be checked regularly and if necessary retightened.

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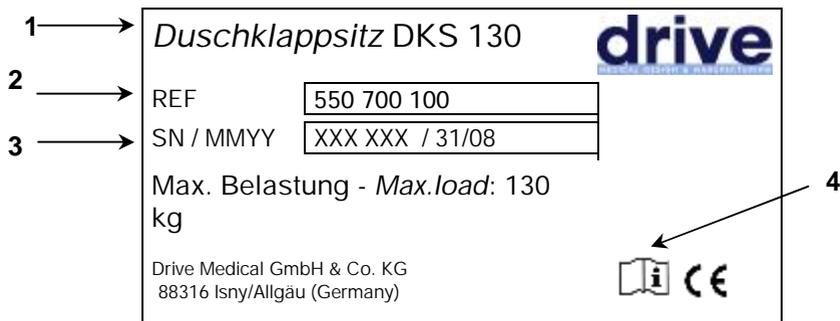
Frame: Welded steel tube, powder coated

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Internet www.drivemedical.de
E-mail info@drivemedical.de

We issue a warranty of 24 months from date of purchase.

*Subject to technical alterations.
Errors and omissions excepted.*

12 / 2008

Drive Medical Limited
Ainleys Industrial Estate
Elland
GB WEST YORKSHIRE HX5 9JP

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Drive Medical GmbH & Co. KG