Gebrauchsanweisung – Operating instructions
Duschstuhl DS 250 ohne Rückenlehne
Shower stool DS250 without back rest
Art.-Nr. – Art. No. 771 100 100

Indikation – Indication

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Shower aids are shower seats, shower stools, shower chairs, shower couches and mobile shower couches which are determined exclusively for the domestic area. Shower aids are considered for people whose physical functional restrictions do not permit the everyday personal care under the shower without other use of aid any more. Shower aids are indicated when the state security of the user is limited. If a decrease of the seat stability (e.g., paralyses in the body area) or a weakness is also given while putting / get up, back support and side armrests must exist if necessary. If necessary the application of shower couch is also necessary.

Lieferumfang – Product contents

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*Shower stool
Operating instructions*

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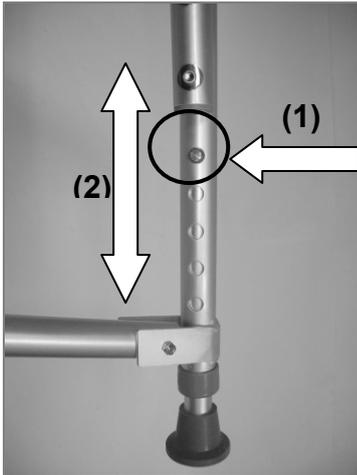
Shower stool is complete assembled for use.

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Make sure that all legs are adjusted at the same height.

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*Do not use the shower stool if it is damaged!
Danger of falling!
Maximum load: 250 kg*

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*Do not use abrasives!
Clean by hand with a damp cloth!*

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We issue a warranty of 24 months from date of purchase.

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Drive Medical Limited · Ainleys Industrial Estate · Elland · GB WEST YORKSHIRE HX5 9JP · www.drivemedical.co.uk

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Drive Medical GmbH & Co. KG · Leutkircher Straße 44 · D-88316 Isny/Allgäu, www.drivemedical.de