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drive
MEDICAL DESIGN & MANUFACTURING

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Dear users,

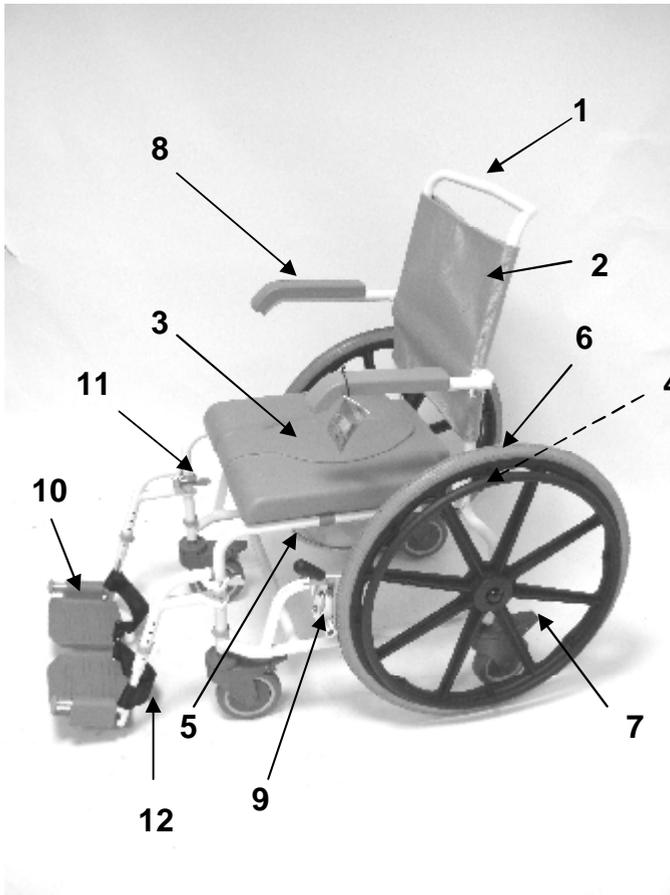
we congratulate on the buying of a technically matured and the best available technology product.

This instructions helps you and the carers as a basis for sure, safe use of the toilet wheel chair and a sensitive use of the product's possibilities.

The instructions describes the preparation, service, use, servicing and cleaning of the product.

This instructions is to be perused carefully and must be always accessible to all users of the product.

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Toilet wheel chairs are necessary when using or achievement of the toilet is not possible because of a handicap. Toilet wheel chairs are also usable as room wheel chairs. Toilet wheel chairs can be pushed over customary toilet bowls.

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This product is suitable for the reapplication.

With the processing the standards of the manufacturer must be considered. This standards are made available on inquiry.

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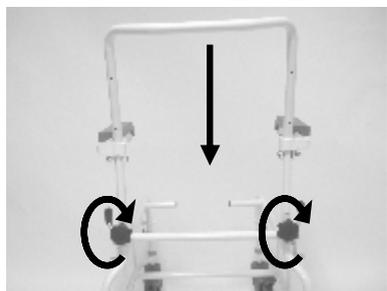
Insert the 4 5 “ wheels in the admission pipe in the frame in the desired position and fix them with the provided bolt clips.

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Put the premounted backrest on the basic rack and protect with the star screws with a clockwise rotation.

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Click in the seat plate as illustrated.

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Pull the backrest cover over the armrests and lock as illustrated

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Insert the bucket from the rear side into the guiding under the seat plate, after use pull out also to the rear side.

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Hang up the provided footrests with both drillings as shown in the respective hold pins

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Insert the hygienic cover in the seat plate, the Shower-Toilet wheel chair is now ready for use.

Montage der Bremse für optionale 24" - Räder – Assembly of the brake for the optional 24" wheels

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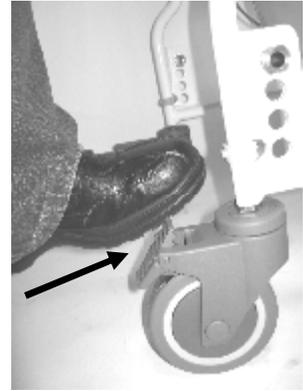
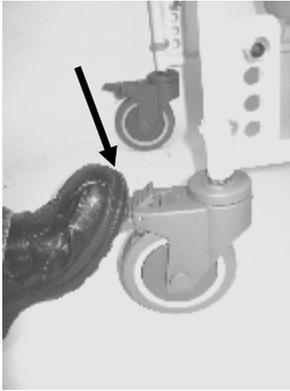
Mount the brakes as illustrated on both sides in such a way, that the the distance between the hung up 24" wheels and the brake is around 1 cm. The 24" wheels have a quick-linchpin for simple assembly. The needed tools are enclosed.

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

*To interlock the brake press lever downwards. **This will lock the direction of the wheel in place.***

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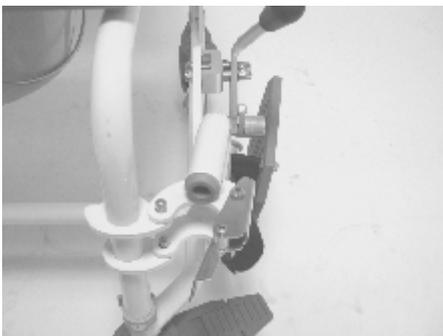
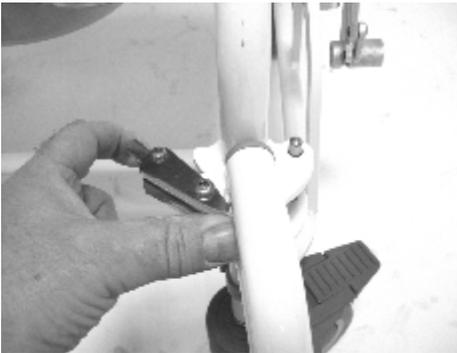
In order to interlock the footrests turn them to the middle until its audible engage.

In order to unlock press the lever as shown inwards.

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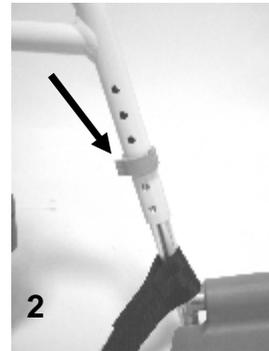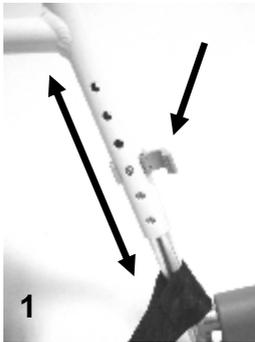To adjust the footrests in the height first unlock the fixing clamp (1). After pinching the fixing bolts the footrests can be shifted (3) in 6 steps in the height for leg lengths from 420 – 500 mm.

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Clean regularly by hand with a damp cloth and usual household cleaning agents!

Use spraying disinfectants p.e. Descosept AF or Bacillol AF.

Do not use any aggressive detergents!

Screws of TRS 130 must be checked regularly and if necessary retightened.

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Frame: Welded alu tube, powder coated

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Internet www.drivemedical.de
E-mail info@drivemedical.de

We issue a warranty of 24 months from date of purchase.

*Subject to technical alterations.
Errors and omissions excepted.*

Drive Medical Limited
Ainleys Industrial Estate
Elland
GB WEST YORKSHIRE HX5 9JP

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Drive Medical GmbH & Co. KG, 88316 Isny/Allgäu (Germany)
Drive Medical GmbH & Co. KG