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Bath seats and bath boards are hung up in the bath or are put on the tub. They have to be mastered by own strength and the lowering is thereby decreased.

They are considered among other things when considerable functional restrictions of the upper and/or lower extremity are. Nevertheless, with application of the aid bathing can be done again widely independently. However, the application assumes that a rest effectiveness enough exists with regard to the body posture and the use of the extremities for the coping with position changes still

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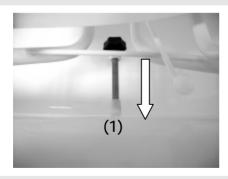
The Swivel Shower Chair is complete assembled for use.

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
Turn both setscrews (1)until they touch the tub wall and prevents slipping of the seat.

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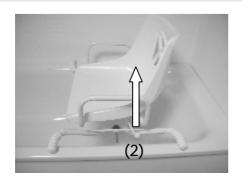The seat can be swiveled after pulling the stop lever (2). This function is to be checked before the first use.

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For climbing out please do it in the opposite way around

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Maximale Belastung: 120 kg

Do not use the Swivel Shower Chair if it is damaged!

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Do not use abrasives!

Clean by hand with a damp cloth!

Product must not be used when surrounding temperature is over 60°C (e.g. use in a sauna etc.)

For cleaning and disinfection use commen products

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Swivel shower chair is suitable for further use after

technical check by a specialized dealer

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